




Quick Start Guide:

# CODMAN® MALIS® CMC® V

## BIPOLAR ELECTROSURGICAL SYSTEM

### System set up:

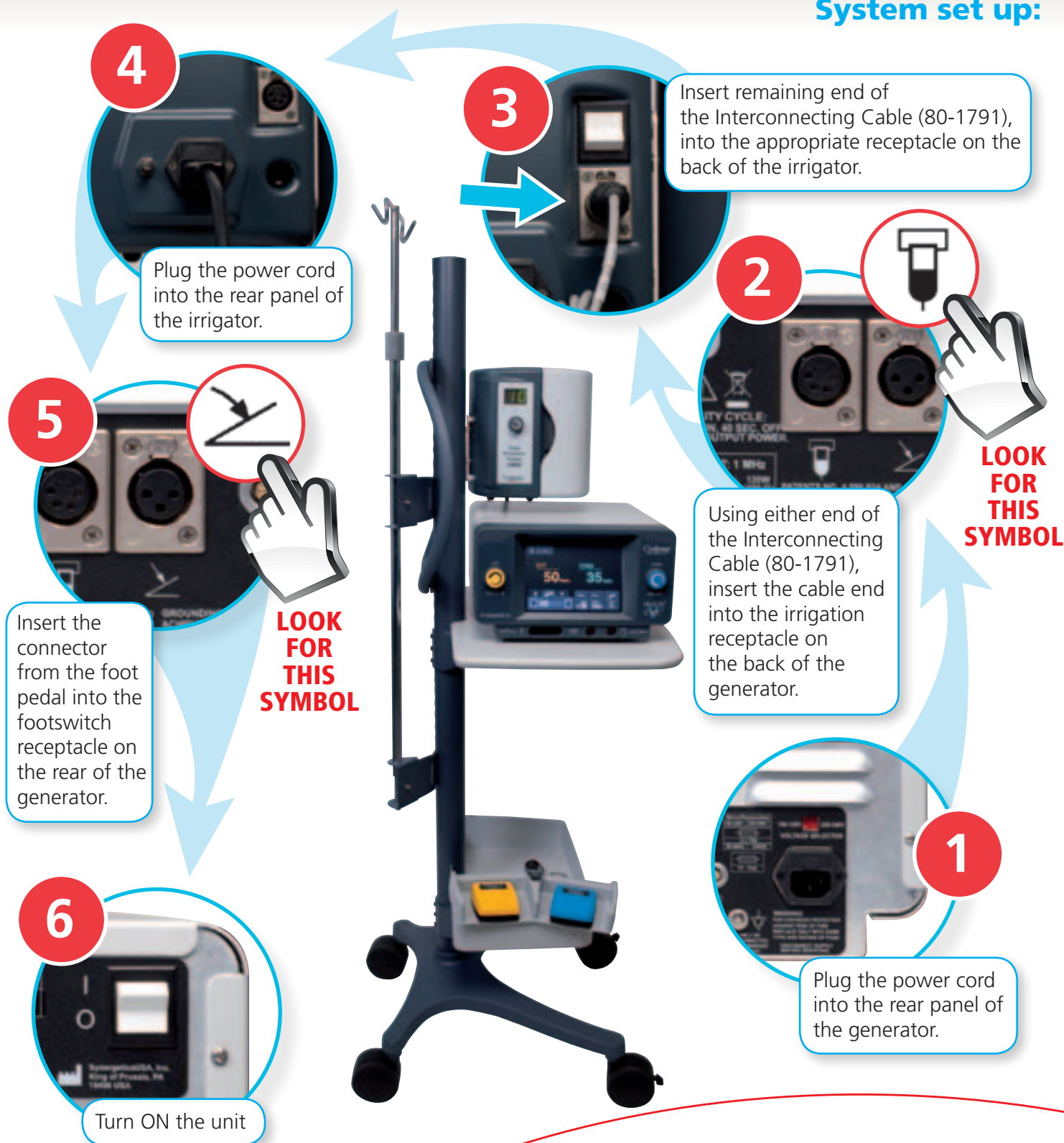

## Operating instructions:

1

Turn the generator ON by toggling the power switch to the "I" position.

2

Wait until the unit has completed the self-diagnostic tests and displays the test status as **PASSED**.

3

Adjust the unit to the desired CUT and COAG settings either by:

- a) Pressing the User Settings Button for pre-programmed settings.
- b) rotating the CUT/COAG rotary knobs.

**PRESS  
HERE**


3a

Press the desired numbered User pre-programmed settings. Once selected the unit will be ready to use.

3b


The unit is ready to use.

Select or edit user

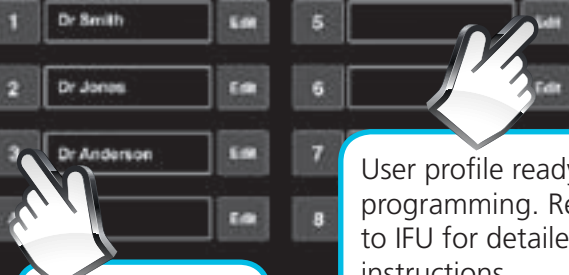

Pre-programmed User Profiles. Press the number button.

User profile ready for programming. Refer to IFU for detailed instructions.

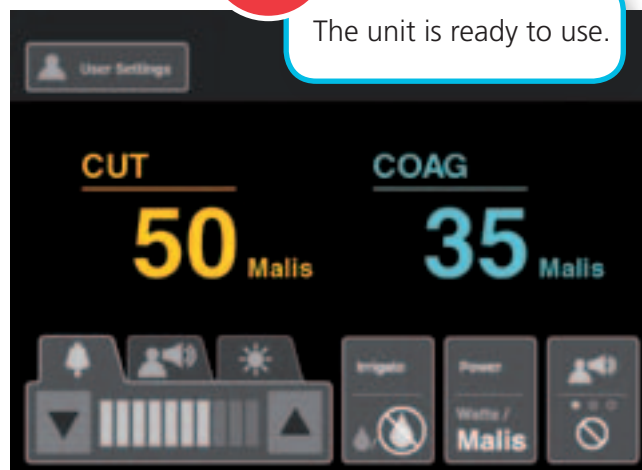

## Coagulation mode:

**1** Use the COAG setting knob to adjust the COAG power setting to the desired level.

**2** To activate the unit in COAG mode, depress the Blue (right) foot pedal.

**3** During COAG activation, the COAG indicator is highlighted and accompanied by a continuous tone.

**4** Release the COAG pedal to stop power delivery.

## Cut mode:

**1** Use the CUT setting knob to adjust the CUT power setting to the desired level.

**2** To activate the CUT mode, depress the Yellow (left) foot pedal.

**3** During CUT activation, the CUT indicator is highlighted and accompanied by a continuous tone.\*

**4** Release the CUT pedal to stop power delivery.

\* Depending on the voice announcement setting, a verbal announcement of the chosen power setting may precede power delivery.

## Adjusting settings during surgery:

1



### Irrigation Button

Press the Irrigation Button to toggle between off and on settings.


2



### Power Unit Display Button

Press the Power Button to toggle between Malis units and watts being displayed.

3

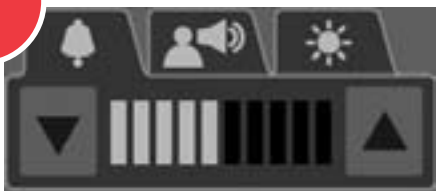

### Alarm Volume Tab

Adjust the alarm volume by using the up and down arrow.

4



### Voice Volume Tab

Adjust the announcement voice volume by using the up and down arrow.

5



### Screen Brightness Tab

Adjust the LCD screen brightness by using the up and down arrow.

6

### Voice Mute Button

Press the Voice Mute Button to toggle between:



Announce CUT/COAG at each activation.



1X Announce once, only when setting has changed.



No voice announcement.

The third party trademarks used herein are the trademarks of their respective owners.

### Codman & Shurtleff, Inc.

325 Paramount Drive  
Raynham, MA 02767  
USA  
Tel: (800) 225-0460  
[www.depu.com](http://www.depu.com)

©Codman & Shurtleff, Inc. 2010. All rights reserved.

ELS-38-000 8/11 BS/RPI

**Codman**  
a Johnson & Johnson company